

ROHO QUADTRO SELECT HIGH/LOW PROFILE

June 1, 2018



Low

High

MODELS/DESCRIPTION

- Quadtro Select features ISOFLO® Memory Control® Unit offers shape-fitting capabilities while the client is seated, allowing quick and easy, on-demand adjustment to maximize function
- Cushion is divided into four sections, which allows for progressive positioning for short and long term changes
- Frequent monitoring of the cushion is required to ensure that proper levels of inflation are maintained

SPECIFICATIONS

- Available in 12-28" widths and 12-20" depths
- Cushions are available in two different cell types:
 - * *Low Profile*: 2" cells
 - * *High Profile*: 4" cells
- Cushions weigh approximately 5 lbs.

Do not overinflate the ROHOs as they are not designed to provide pressure relief when they are fully inflated. The general rule is to allow only a finger's space (1/2") between the buttocks and the base of the cushion.

ELIGIBILITY

- Must meet general eligibility requirements
- Client requires the use of a wheelchair for mobility and will use the cushion as an accessory to the wheelchair.
- SIP/SHP/FHB clients may be eligible for a cushion without the use of a wheelchair – see cushion general policies

REQUISITIONING AUTHORITY

- Physiatrist
- Plastic Surgeon
- Licensed Occupational Therapist and/or Physical Therapist
- Home Care Nurse (in consultation with an Occupational Therapist or Physical Therapist)

ORDER DETAILS

- Specify size required
- Order in even-numbered width by depth dimensions (i.e. 16" x 16", 18" x 18", etc.) - odd-sized cushions are not available.

Note: Roho cushions are repairable! Please do not order a replacement Roho cushion if only a repair is required. SNE Technicians will repair cushions and if deemed unrepairable a replacement will be provided.

FORMS

- [Special Needs Equipment Requisition](#)