

ROHO ENHANCER

June 1, 2018



MODELS/DESCRIPTION

- The Enhancer is a dual-valve system for midline channeling of the femurs, lateral stability and tissue protection
- Recommended for enhanced pressure distribution, positioning and posture
- Frequent monitoring of the cushion is required to ensure that proper levels of inflation is maintained

SPECIFICATIONS

- Available in 12-20" widths and 12-20" depths
- Contoured cushion containing a combination of low (2") and high (4") profile cells.
- Cushions weigh approximately 4 lbs.

Do not overinflate the ROHOs as they are not designed to provide pressure relief when they are fully inflated. The general rule is to allow only a finger's space (1/2") between the buttocks and the base of the cushion.

ELIGIBILITY

- Must meet general eligibility requirements
- Client requires the use of a wheelchair for mobility and will use the cushion as an accessory to the wheelchair.
- SIP/SHP/FHB clients may be eligible for a cushion without the use of a wheelchair – see cushion general policies

REQUISITIONING AUTHORITY

- Physiatrist
- Plastic Surgeon
- Licensed Occupational Therapist and/or Physical Therapist
- Home Care Nurse (in consultation with an Occupational Therapist or Physical Therapist)

ORDER DETAILS

- Specify size required
- Order in even-numbered width by depth dimensions (i.e. 16" x 16", 18" x 18", etc.) - odd-sized cushions are not available.

Note: Roho cushions are repairable! Please do not order a replacement Roho cushion if only a repair is required. SNE Technicians will repair cushions and if deemed unrepairable a replacement will be provided.

FORMS

- [Special Needs Equipment Requisition](#)