

ROHO NEXUS SPIRIT

June 1, 2018



NOTE: as of December 2021, the Nexus Spirit has been discontinued by the manufacturer. We will still issue these as recycle stock is returned to us.

MODELS/DESCRIPTION

- Nexus Spirit provides the stability of a contoured foam base and a Roho cell insert
- Allows for increased stability for transferring and positioning of the pelvis and lower extremities for enhanced sitting posture
- Frequent monitoring of the cushion is required to ensure that proper levels of inflation is maintained

SPECIFICATIONS

- Available in 14-18" widths and 14-18" depths
- 20" wide available with 18" depth only
- No custom sizes are available
- Cushions weigh approximately 3 lbs.

Do not overinflate the ROHOs as they are not designed to provide pressure relief when they are fully inflated. The general rule is to allow only a finger's space (1/2") between the buttocks and the base of the cushion.

ELIGIBILITY

- Must meet general eligibility requirements
- Client requires the use of a wheelchair for mobility and will use the cushion as an accessory to the wheelchair.
- SIP/SHP/FHB clients may be eligible for a cushion without the use of a wheelchair – see cushion general policies

REQUISITIONING AUTHORITY

- Physiatrist
- Plastic Surgeon
- Licensed Occupational Therapist and/or Physical Therapist
- Home Care Nurse (in consultation with an Occupational Therapist or Physical Therapist)

ORDER DETAILS

- Specify size required
- Order in even-numbered width by depth dimensions (i.e. 16" x 16", 18" x 18", etc.) - odd-sized cushions are not available.

Note: Roho cushions are repairable! Please do not order a replacement Roho cushion if only a repair is required. SNE Technicians will repair cushions and if deemed unrepairable a replacement will be provided.

FORMS

- [Special Needs Equipment Requisition](#)